

# Male Urinary Incontinence Sheath – Instructions for Use

---

## Application

1. Wash and dry your penis before use and remove any pubic hair.  
**Do not use talc or creams.**
  2. Remove the protective film from one side only of the double-sided adhesive strip.
  3. Arrange the coiled strip around your penis and remove the second film.
  4. Place the sheath over your penis, **leaving a space between the glans and the sheath tube** to favour urine outflow.
  5. Unroll the sheath over the adhesive strip and **gently press** to ensure it adheres properly.
  6. Connect the sheath to the urine drainage bag.
- 

## Removal

Carefully roll up the sheath to make removal easier.  
Then remove the double-sided adhesive strip.

---

## Warnings

- **Do not use the product if you have a known latex allergy.**
- It is recommended to **replace the sheath every 24 hours.**
- **Thorough local hygiene** prior to application is essential.
- **Do not overtighten** the adhesive strip to avoid circulation problems.
- Product is **non-sterile, non-sterilisable.**
- Recommended for use in combination with the adhesive strips.
- Keep out of reach of children.

Any serious incident resulting from the use of this medical device must be reported to the manufacturer and the competent authority of the Member State where you reside.

---

## Disposal

Dispose of the medical device in compliance with local waste disposal regulations.  
**It does not contain toxic substances.**